

Table of Contents

Warningspg 2

Section I: Product Description & Photo

Starlet Flat.....pg 3
Starlet Tiltpg 3
Starlet Deluxe..... pg 4

Section II: Directions for Use

How to Adjust Height of the Table pg 4
How to Adjust the Backrest pg 4
How to Adjust the Legrest pg 5
How to Install the Headrest..... pg 6
How to Use the Wheel pg 7

Section III: Troubleshooting

The Table will not Change Heightpg 7
Diagnose and Find the Problems.....pg 7
Foot Control Replacement Instructionpg 9
Motor Actuator Replacement Instructionpg 10
Control Box Replacement Instructionpg 11
Pneumatic Pump Replacement Instructionpg11
The Back-rest or Leg-rest will not Lock into Position.....pg 12

Section IV: How to Care for Your Upholstery.....pg 12

Warnings:

Improper use of this device may cause injury.
Be sure to read all operating instructions prior to use.

Check the padding and upholstery when unpacking the table. Do not use it if there are signs of damage. (Our warranty excludes damage caused by improper set-up, accident, abuse, or use other than for the intended purpose)

Weight limit (including client and accessories):

The “Starlet Flat” Lifting Capacity: 450Lbs/204kgs

The “Starlet Tilt” Lifting Capacity: 450Lbs/204kgs

The “Starlet Deluxe” Lifting Capacity: 450Lbs/204kgs

Be certain that the table is properly lowered, without any tilt being present, before supporting a patient.
Otherwise, the patient may lose balance and fall.

Electrical Shock Danger:

The power supply/control box is located below the table. For safety purposes, do not touch the power supply and control box when the table is in operation. Complete all the preparatory work well before connecting the plug to the power. Please refer to the instructions carefully before using it.

Do not put excessive weight or pressure on the Headrest or Table Extender.

Do not expose the product to the extreme moisture or temperature.

These table weighs up to 159.7Lbs/72.6kgs.

Do not turn the unit on its side without using proper safety procedures.

Under any circumstance, forbid going under the table or scissors construction in case of any Personal injuries caused by the collapse of the table.

Keep children and pets away from the massage table, Potential electric shock accidents or personal injury may occur if there is interference with the scissor action.

Do not leave or storage any item inside the scissors construction of the table or underneath the table top.

PRODUCT DESCRIPTION & PHOTO

Item: Starlet Flat


Item: Starlet Tilt



Switch




Gas pump for backrest

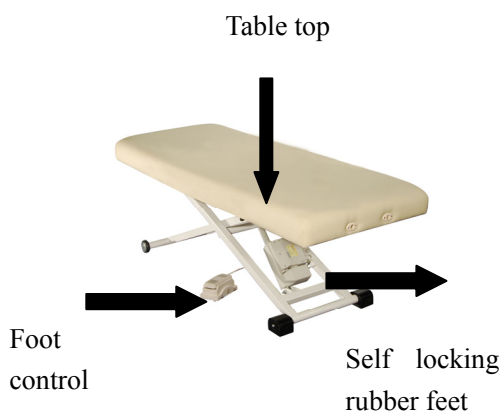

Removable
armrest

Control box
Foot control

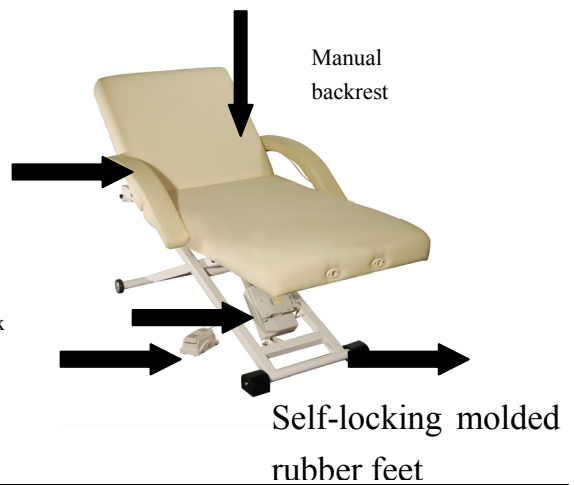


Lifting Capacity:	450lbs/204kgs
Weight :	121lbs/55kgs
Length:	72"(185cm) 75"(193cm)
Height range:	18"-35"(46-102cm)
Width:	28" 30" all available
Padding	3"/7.6cm soft foam thickness
Upholstery	PU or PU available
Accessories:	Including adjustable headrest, lowered armrest

Lifting Capacity:	450lbs/204kgs
Weight:	138.6lbs/63kgs
Length:	72"(185cm) 75"(193cm)
Height range:	18"-35"(46-102cm)
Width:	28" 30" all available
Padding	3"/7.6cm soft foam thickness
Upholstery	PVC or PU available
Accessories:	Including adjustable headrest, lowered armrest

Item: Starlet Deluxe

Lifting Capacity:	450lbs/204kgs
Weight :	159.7lbs/72.6kgs
Length:	72"(185cm) 75"(193cm)
Height range:	18"-35"(46-102cm)
Width:	28" 30" all available
Padding	3"/7.6cm soft foam thickness
Upholstery	PU or PU available
Accessories:	Including adjustable headrest, lowered armrest

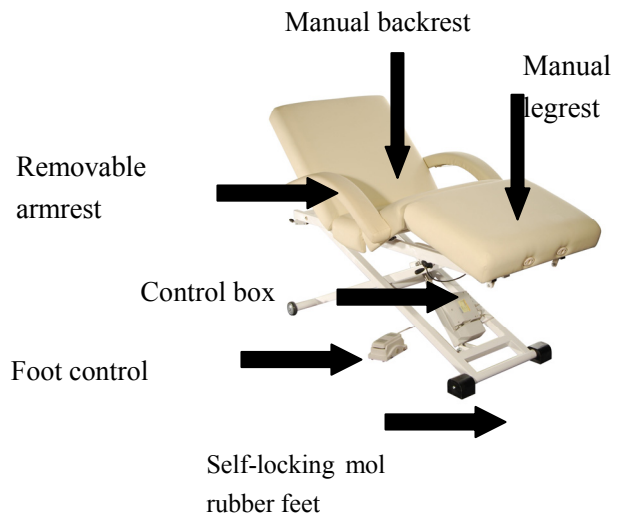

Gas pumps

SWITCH

DIRECTIONS FOR USE

How to Adjust Height of the Table

The Starlet Flat /Starlet Tilt /Starlet Deluxe table comes with a foot-control to operate the table height and gas spring to operate the angle of the backrest and legrest.

1)Foot-control:


Operate the foot-control as shown below to raise or lower the height of the table. When operating the table's controls, be sure to observe all cautions and warnings as given in the manual.



How to Adjust Backrest

How to Raise the Backrest

To raise the backrest from a flat position, pull the switch outwards with your fingers until the desired angle is reached. The backrest will automatically lock into this position when you release your hold. To increase the angle, merely repeat the action until the top is in the desired position.

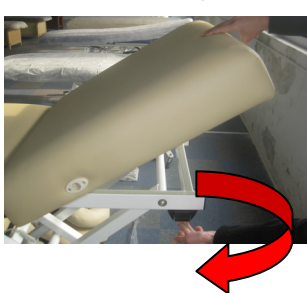


GAS PUMP



Lock for the backrest

How to Lower the Backrest

To lower the backrest, pull the switch outwards slightly and guide the backrest down with a firm power. Release the switch when you have the backrest at the required level and the backrest will lock automatically in position.

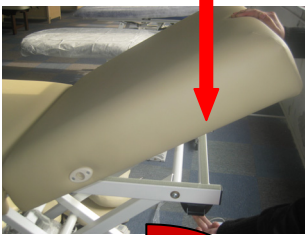

Lock for the backrest



Gas pump

How to Adjust the Legrest

The leg-rest on the Starlet Deluxe table is also manually adjustable.

How to Raise the Legrest (Starlet Deluxe):

To raise the legrest from a flat position, pull the switch outwards with your fingers until the desired angle is reached. The legrest will automatically lock into this position when you release your hold. To increase the angle, merely repeat the action until the top is in the desired position.



Gas pump



Switch


How to Lower the Legrest:

To lower the legrest, pull the switch outwards slightly and guide the legrest down with a firm power. Release the switch when you have the legrest at the required level and the legrest will lock automatically in position.





Gas pump

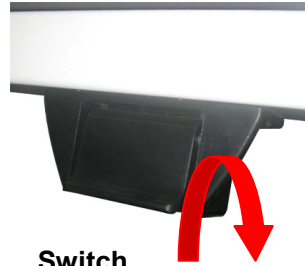

Switch

How to Install the Headrest

Adjustments are easily made by loosening the cam-lock, adjusting the height and angle, then locking the cam-lock.

Cam-lock



Adjustment are easily made by loosening the cam-lock. Adjust the height and angle, and then lock the cam-lock



Plug the face rest into the nylon tube.




plug it to the bottom and lock the cam-lock.

Pay attention to: The length of the excess bar should be at least 4".

How to Use the Wheels

The Starlet Flat/ Starlet Tilt/Starlet Deluxe table is equipped with self-locking molded rubber feet and wheels.

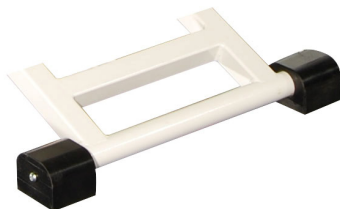

① Self-locking molded rubber feet



② Wheels


Place the table in the location where you prefer and the self-locking molded rubber will stable the massage table. When moving the massage table, please raise the self-locking molded rubber feet and move to the place you prefer and lower down to the ground.

Troubleshooting

The Table will not Change Height:

Diagnose and Find the Problem:

The motor structure as below

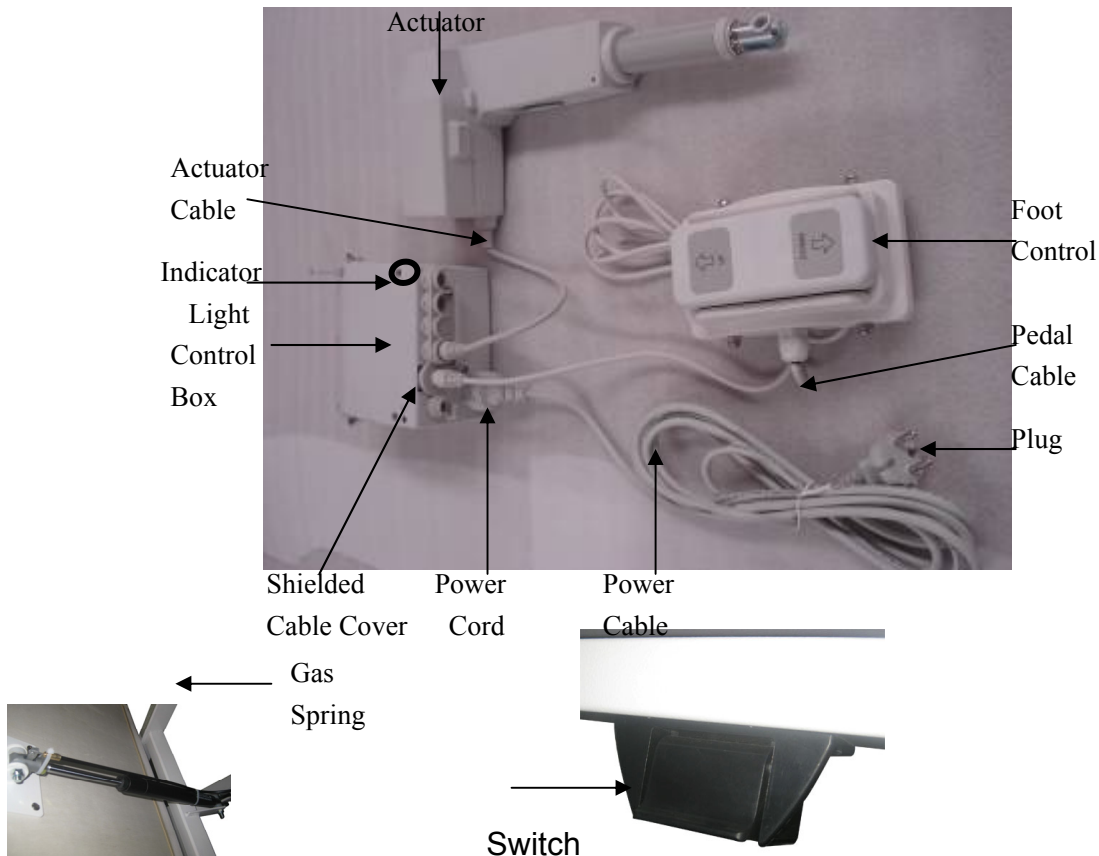

1. The indicator-light is not on:

- ◆ Check the outlet to be sure that it has power supply and the cable is plugged in

Note:

Make sure that all male and female connectors are firmly pushed into each other. Even a slight misconnection can cause a malfunction.



1st step: unplug the power cable from the wall outlet



2nd step: **Get a new power outlet which has a definite power supply, plug the power plug into the new power.**

TROUBLESHOOTING

◆ Check the control-box to make sure that it has power and the cable is plugged in..



Good connection



Poor connection



Unplug the power cable from the control-box, then plug in again.

Result: If the indicator light is still not on after carrying at the above process, it means the control box is damaged and need replacement.


2. The indicator light is on, but the table still can not change height:



CAUTION

The table weighs as much as 72.6kg.
Use extreme caution if turning the table on its side. Seek assistance to ensure safe operation!

◆ Move the table to a quiet area, try the foot-control. Listen: both the control-box and the foot-control should sound “click” when you step on the foot-control.

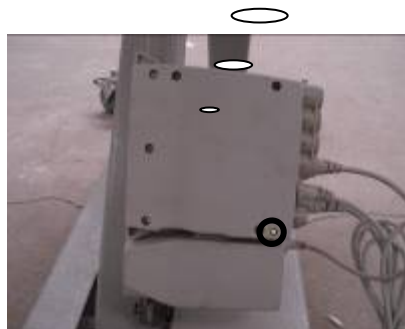

Keep power on, the control-box should “click”, when you step on the foot-control.



Keep power on, the foot-control should “click”, when you step on the foot-control.

TROUBLESHOOTING

◆ Press the foot-control: if there is no corresponding click within, it means the Foot-control is damaged and will need to be replaced. (Refer to manual “Replacement Foot-control instructions”)

Result1: If the power is on and the indicator-light is on, but the control-box does not “click” when you press the foot-control, firstly ensure the foot-control-cable is properly connected to the control box. If the connection is solid, it means the control-box is damaged and need replacement. (Refer to manual “ Control Box Replacement Instruction”)



The foot-control-plug is at the end of the cable. The socket is on the control-box.



Good connection

Result2: If the table will still not change height after carrying out the above processes, the indicator is on, both control-box and foot-control “click” when you press the foot-control and all cables have a solid connection, it means that the actuator is damaged and needs replacement.

PS: You will have a 3 years’ guarantee for the actuator (including the Motor Actuator and the Control-box) from the date of purchase.


Please contact the retailer or wholesaler to claim a new part if the table does not work after the above diagnosis.

Foot Control Replacement Instruction

Replacement of the foot-control will be necessary if the foot control does not activate the elevation function.

To Replace the Foot Control, Follow these steps:

1.The foot-control is connected directly into the control box by foot-control-cable, remove the shield cable plug from the control-box and remove the foot-control-plug from the foot-control-socket by pulling them firmly until fully disconnected.

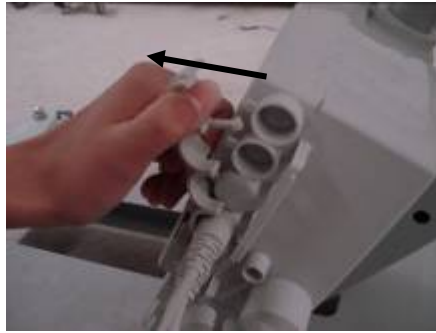

Remove the shielded cable cover



Unplug the foot-control-plug

2. Obtain the new foot -control.
3. Install the new foot-control by connecting the control's plug to the control box socket and re-cover the shielded cable plug.

Motor Actuator Replace Instruction

1. Disconnect the power cord from the power outlet.

Lock the table's four wheels, carefully turn the table on its side. As a safety precaution, it's highly recommended that assistance by another person is utilized when turning the table on its side.



Turn the table on its side



Remove the shielded cable cover

2. Remove all the cables from the control-box, loosen and remove the screw.



Remove all the cables from the control-box



Loosen and remove the screw using a screw-driver

3. Remove the control-box from the actuator by sliding it upward.
4. Remove the clamp spring by forceps at both end of the actuator.
5. Remove the actuator form the table.



The clamp-spring
at both-end of the actuator



<1> Slide the control box upward to split
it from the actuator



<2> Remove the clamp spring and the bolt



<3> Remove the actuator from the table

6. Obtain the new actuator.

7. Install the new actuator using these instructions in reverse.

Control Box Replacement Instruction

Refer to manual "Motor Actuator Replacement instruction" step 1.2.3

1 Obtain the new control-box.

2 Install the new control-box.

Pneumatic Pump Replacement Instruction

1. Remove the clamp spring from both the end of pneumatic pump with a spanner..

2. Obtain the new pneumatic pump.

3. Install the new pneumatic pump.




When the backrest or leg-rest can not lock into position

When the backrest or leg-rest can not lock into position, due to the pneumatic pump' damage, it should be replaced as follows:

1. Please use a spanner to loose the screws both on the end of pneumatic pump.

2. Remove the actuator from the table

3. Install the new pneumatic pump.

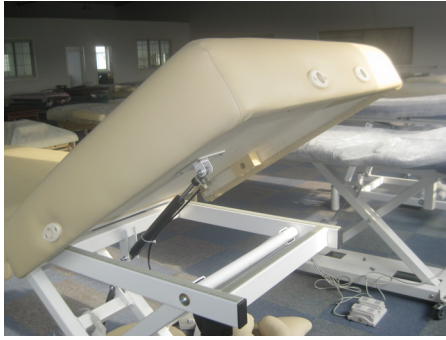

How to Care for Your Upholstery Fabric

Congratulation on your table purchase. In order to maintain your products' quality and ensure the life of your table, we have tested several preventative steps you can take to protect the upholstery on your table and accessories.

Here are a few suggestions in caring for your upholstery fabric:

- Avoid storing your table in extreme cold temperatures since excessive coldness will cause cracking. So please do not leave your table in unheated areas during cold weather.
- Avoid storing your table in extreme heat temperatures since excessive heat will cause the upholstery to stretch and pucker. So be sure your table is away from heaters, electrical appliances, direct sunlight and the inside of your car during hot weather.
- Protect your table and accessories with sheets and fleeces. We offer three-piece 100% Cotton Flannel Sheet Set and Fleece Sheet Set to guard against oil stain whilst providing total comfort for your clients.
- Clean your upholstery fabric everyday
We recommend cleaning your upholstery fabric with a mild detergent to protect it against oil stains and dirtiness, which will cause cracking and permanent stains over time. Common practice is to mix small anti-bacterial dish soap with water in a spray bottle and wipe down the table followed by a clean water.
- Use both ends of the table. Occasionally, switch the face rest from one end of the table to the other to avoid uneven wear.
- Protecting your table from general abrasions, cuts and punctures



Caution:

1. Rubbing alcohol will dry out the vinyl causing it to crack.
2. Prolonged exposure to temperature extremes may cause damage to vinyl.
3. The moist weather will shorten the actuator life, keep it away from water.