

Therapist's Choice® Massage Table Warmer

INSTRUCTIONS & WARNINGS

(Read before use and retain for future reference)

Safety Information

WARNING: In order to reduce the risk of fire, electrical shock, and/or injury to persons when using the electrical appliance, it is important to follow these basic safety precautions:

- Do NOT use it on your Bed to sleep with! This product is NOT a Bed Warmer/blanket. This is NOT a heater. It is intended for use ONLY as a Massage Table Warmer for use under the constant supervision of a healthcare professional. Property Damage, Fire, Injury and Death can result if not used as intended.
- Never use if wet. Do not allow the table warmer, control unit or power cord to come into contact with liquids.
- Not intended for use with pets.
- Check table warmer frequently to ensure it is NOT bunched or folded as this could cause excess heat and result in injury or fire. Must be unfolded completely while in use.
- If your table warmer does not function properly, discontinue use immediately and consult the manufacturer or distributor.
- Prior to each use: Inspect the cord, plug, power supply and table warmer for damages. If damaged, discontinue use immediately and consult the manufacturer or distributor.
- If using an adjustable height massage table, make sure the cord cannot be caught.
- Table warmer should NOT hang over the edge of massage table.
- Do not use with disabled persons, infants, young children or people with insensitive skin.
- Long applications on the high setting can result in skin irritation or burns.
- Keep control unit away from open windows.
- Do not pierce with sharp objects.
- Use table warmer only on an alternating current supply circuit of correct voltage.
- Keep the flexible power supply and switch (as much as possible) off the top of the table & don't tuck it under the pad.
- Unplug the unit at the power outlet when not in use.

NOTE: The electrical specifications of the table warmer are:

- North America (Fahrenheit degree): 110VAC, 70W, 60Hz

Care & Maintenance

Table Warmer - Care Instructions:

CAUTION: To avoid damaging your table warmer, carefully read and follow the instructions below.

Spot Cleaning Instructions:

1. Before cleaning, ensure the control is set to "Off" and the plug is removed from the wall socket.
2. Sponge the soiled area using lukewarm water and mild detergent or soap.
3. Follow with a sponging of clean water. To remove excess water, compress the wet area between dry absorbent towels.
4. Make sure Table Warmer is completely dry before using.

Storage Instructions:

- Fold neatly and loosely, without sharp bends in the heating element.
- DO NOT place objects on top of you massage table warmer.
- DO NOT ROLL.

Operations & Controls

1. Plug into wall outlet
2. Flip Switch to "ON" Position
3. Press "High" button for Pre-Heat Setting
4. Press "Temp" button to adjust Temperature Setting
5. Press "Time" button to adjust timer
6. When finished or unattended, Flip Switch to "OFF" position and unplug from wall outlet.

ON/OFF Switch: Turns device ON and OFF

On Start-Up: The Stand by state displays as Temp 0 and Timer 0

Temp button: Circulates from L, 1, 2, 3, 4, 5, 6, 7

Time button: Circulates from 1-9 hours. Note: you have to first press the Temp or High button for the Timer Button to Function.

High button: Displays "H", which is a preheating setting. It stays on H for 30-60 minutes and then automatically switches to the "L" setting.

Troubleshooting

Faulty Indicator:

- If temperature displays as **P**: Please check that the coupling plug of the electric blanket is connected firmly and then restart it. If faulty indication cannot be removed, this means internal connections of the electric blanket may have problem.
- If temperature displays as **E**: This means electric blanket is broken and most likely the result of an internal short circuit.

How To Test the Table Warmer:

You can test it with a thermometer, but the most effective way to test the warmer quickly is to roll up your table warmer, set it to high heat and wait 5 minutes. Place your hand in the center. If you feel heat, it's working.

My Table Warmer Will Not Heat Up:

The table warmer provides warmth within 5 minutes of a client laying on it. Adding a blanket will help insulate and retain the heat between services. The table warmer itself is not designed to retain the heat.

My Table Warmer Is Not Getting Hot Enough:

The table warmer is designed to provide plenty of warmth, but only when a client or blanket is laying on it will it retain the heat. This is not a blanket warmer, it's made to take the chill of the table and to provide warmth to the client.

My Table Warmer Shuts Down:

The table warmer automatically shuts down when the Timer setting ends. Make sure the timer is set to a desired time. Note that the "H" temperature setting automatically changes to L after about an hour.

My Warmer Won't Turn On:

If the display no longer turns on, the table warmer could be plugged into a faulty outlet, or the overheat safety circuit could have been activated. Make sure the pad is plugged into a tested outlet, and if the display is still not working, the table warmer needs to be replaced or returned.

